DILAGENT®

REF A.1004

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Silicone dilator for pelvic floor

The package contains:

- one biomedical silicone nozzle with pressure valve, safety disk and approx. 60 cm tube
- one 60 mL syringe
- one sachet of **Dilatan Cream**
- one sachet of **Sapigen K** intimate cleanser
- user instructions

PROCTOLOGY

INDICATIONS

DILAGENT is indicated for the treatment of anorectal diseases caused by a hypertonic sphincter, namely anal fissures, haemorrhoids, anal pain and painful spasms after surgical treatment of the anorectal segment. Excellent results have also been obtained for post-surgical stenosis of the anus and anal canal, for paradoxical contraction of the puborectalis muscle (anismus) and for rehabilitation after sphincter repair surgery.

HOW TO USE THE PRODUCT

Before using DILAGENT:

- remove all the air contained in the probe by inserting the syringe into the relief valve and suctioning it
- perform a warm hip bath for 5 minutes (fill the bidet with warm water and wash the anal region with **Sapigen K** to fully relax the perineal muscles.

The recommended position for nozzle insertion is left lateral decubitus, which is also known as the Sims position. (Fig.2)

Alternatively, the probe can be inserted with the patient seated on the bidet.

Fill the syringe with 50 ml of warm water with temperature in the range of 40° and 45°C (domestic hot water). Check the temperature with a thermometer.

Lubricate the nozzle and gently insert it into the anal canal until the handle touches the anus. Slip the DILAGENT tube between the legs and grasp the pressure valve with the fingers (Fig.1). Take the syringe that was previously filled with 50 ml of warm water and connect it to the pressure valve.

Note: during phases of acute pain and bleeding, as in the case of acute anal fissures, the introduction of the probe can cause discomfort; therefore, we recommend injecting a few ml. of water for the first 2 or 3 days.

The maximum quantity of water that can be inserted into the probe is 50 ml. A larger amount of water might tear the walls of the nozzle.

The method of use varies depending on the disease to be treated (see the **Treatment Plan below**).

Note: the particular shape of DILAGENT allows the user to walk about the house; hence, the patient is not obliged to remain lying in bed or seated on the bidet. However, limited movements are recommended to avoid injury or damage.

MECHANISM OF ACTION

DILAGENT is gradually filled with warm water after being inserted into the anal canal. Its principal feature is the ability to adjust the balloon size as required

based on the symptoms. The particular shape of the balloon ensures transfer of heat not only to the anal canal (specifically the internal and external sphincter), but also to the puborectalis muscle and smooth muscles of the rectal ampoule, which are always involved in anal hypertonia.

The shape of the distal end enables the balloon to firmly anchor to the puborectalis muscle; therefore, it is not dislocated during movements (Fig.3). The transfer of heat to the entire sphincter system allows to reduce muscle hypertonia with subsequent improvement in or relief of pain.

TREATMENT PATTERN

Anal fissures, haemorrhoids, pain, anal spasm, anismus, anal stenosis.

Treatment must have a duration of 3-4 weeks. The quantity of water inserted into the balloon must be 25 ml or more. Treatment for anal stenosis can commence with just 10 mL of warm water. The gradual reduction in pain during treatment must correspond to an increased introduction of water into the balloon per session. The introduction of 40-50 mL of water during the third and fourth week of treatment is to be hoped.

During the first week, inject the balloon with the quantity of water required to feel warmth in the anal canal. The quantity of water must be based on the symptoms. The injection of larger quantities of water increases balloon dilation, which is an essential condition for healing.

The recommended 4-week treatment pattern is specified below:

- before each treatment session, fill the bidet with warm water and perform a hip bath for 5 minutes;
- for the first two weeks, apply DILAGENT 2-3 times a day (morning, afternoon and evening), 12-15 minutes per session;
- during the two subsequent weeks, reduce to two applications a day (morning and evening), maintaining the duration of 12-15 minutes per session.

Rehabilitation of the sphincter

After sphincter repair surgery due to faecal incontinence, DILAGENT can be used to perform active exercises that

encourage recovery of sphincter tone.

After inserting the balloon into the anal canal as described above, introduce 20-25 mL of warm water with the supplied syringe. Start contracting the anal sphincters at regular intervals. Voluntary contractions must last 2 seconds with 4-5 second intervals. Three sessions a day, 10 minutes each, are recommended.

Note: avoid over-exercising because it can cause muscle fatigue and increase incontinence.

At the end of the session, deflate the balloon by suctioning the water with the syringe through the relief valve. Grasp the handle of DILAGENT and remove it very slowly. Then wash the anal region thoroughly with warm water and Sapigen K intimate cleanser. Dry with a soft cloth without using toilet paper. Rinse off DILAGENT under a continuous jet of warm water and soap the probe with Castile soap. Dry with a cloth and store DILAGENT in a dry place.

For details about the correct use of DILAGENT and for any information, ask the general practitioner or your trusted proctologist.

CONTRAINDICATIONS

There are no contraindications to the use of DILAGENT.

UNDESIRABLE EFFECTS

During treatment, the patient might experience rectal tenesmus (constant urge to evacuate), anal itching and a damp feeling in the anus. These symptoms are temporary and clear at the end of the session with DILAGENT. During the early part of the treatment, especially the first few days, the pain may increase during the first minutes of the treatment. In this case, the patient is recommended not to suspend therapy but to take an analgesic that, by reducing the pain, will allow

treatment with DILAGENT to continue.

If the pain becomes so intense as to prevent continuation of treatment, suspend the application of DILAGENT and contact the prescribing specialist.

SPECIAL WARNINGS

DILAGENT is a medical device and it must only be used as physical treatment for the aforementioned diseases.

The patient is strongly recommended not to use the device for any other purposes. Keep away from children. Do not freeze. Do not boil. Do not warm in the microwave oven. Do not cut. Do not ingest. Do not disperse in the environment after use. Only use neutral detergents to wash the DILAGENT probe. The syringe does not need frequent washing. The use of DILAGENT is for single patient use only.

With the cooperation of Dr. Piercarlo Meinero

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